


To Install the special USB-Driver for the Sphinctometer.

1. Please connect the USB cable with the Sphinctometer and The PC, and store the special drivers on your PC (we chose : D :\Driver USB msm PoMedico V 13). Then turn on the Sphinctometer.


2.

Open the: Device Manager (Gerätemanager) And look for not detected Device (yellow symbol)

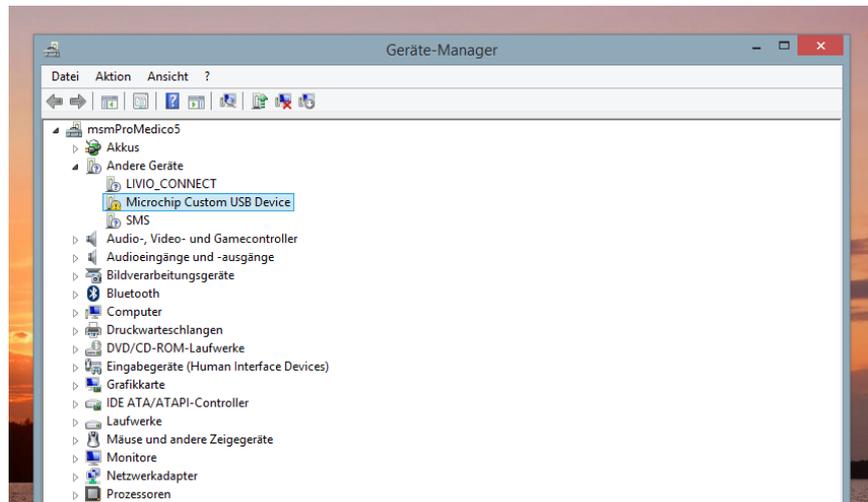


3.

Open the special Dialog with a click on the right mouse button, after chosen the yellow Symbol with the mouse cursor. Then click on properties (Eigenschaften)

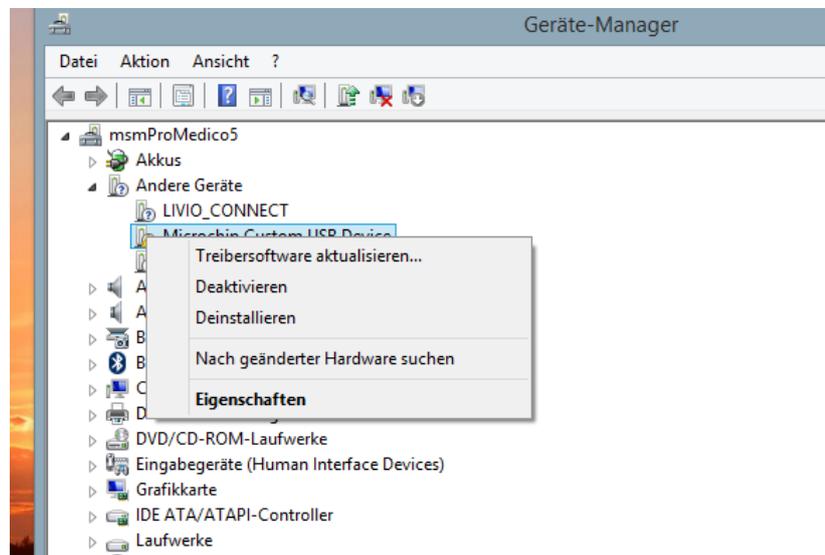


4.

Klick on Drivers  
(Treiber)

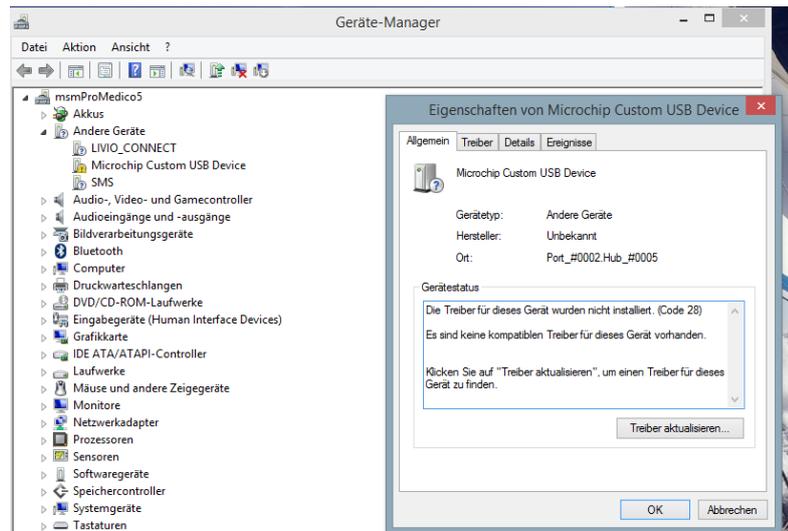

5.

Klick on renew driver  
(Treiber aktualisieren)

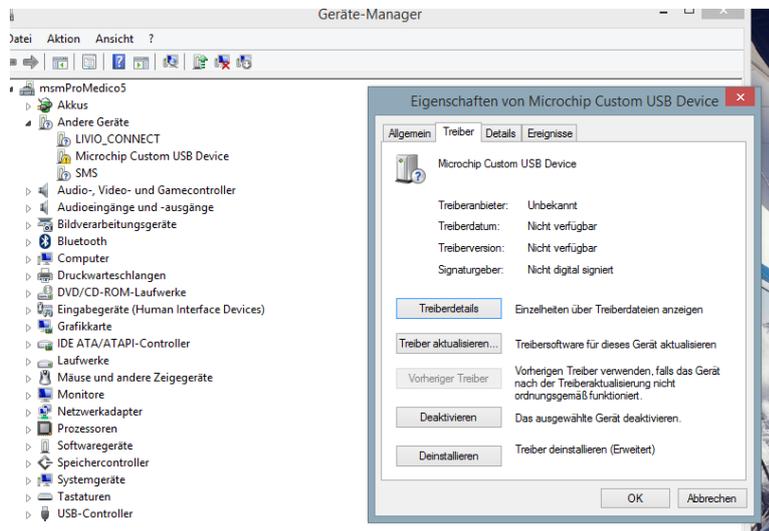


6.

Klick on


Search on the computer  
after driver software...

(Auf dem Computer  
nach Treibersoftware  
suche. ....)

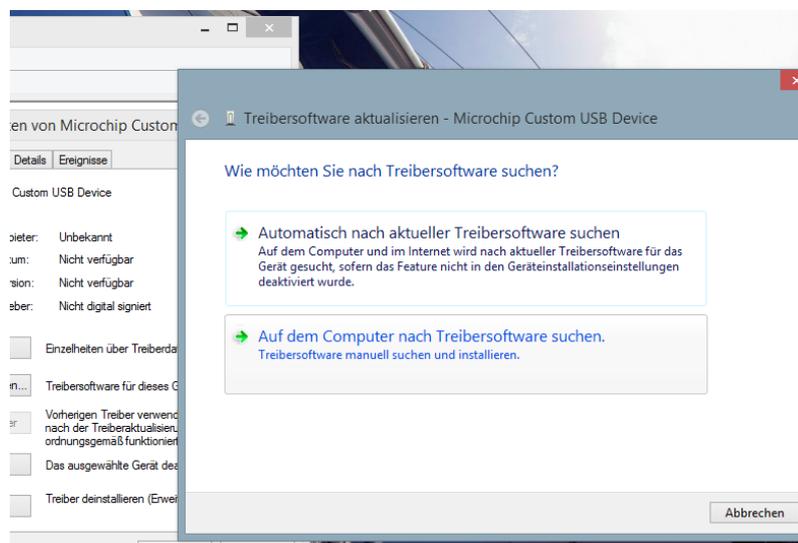


7.

Search the directory where the drivers are stored. Here D : \Driver USB msm PoMedico V 13

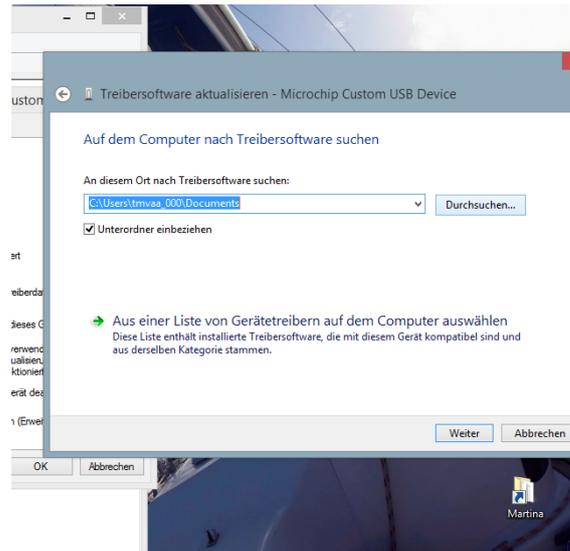


8.

Chose the Directory and click on OK.

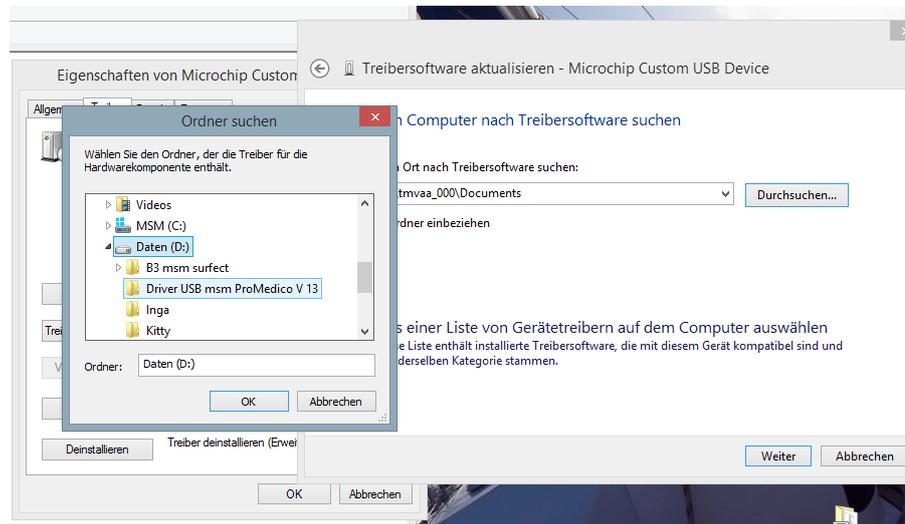

9.

The driver will be installed. The yellow symbol disappeared.

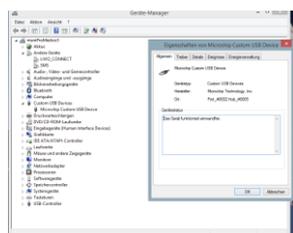

Now you can work.

